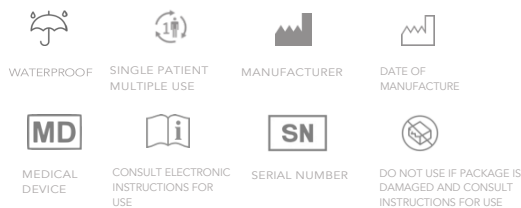
INTENDED USE: This medical device is intended for immobilization of a patient's limb. As an orthopedic medical support splint, it is used to heal bones, muscles or ligaments by immobilizing and supporting the limb during a specific period of time.

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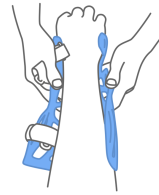
- 1 -

Expand the splint to place it on the arm of the patient
 Hold either side of the splint with each hand. Gently pull on both sides. Insert the arm of the patient in the splint, starting with the thumb.

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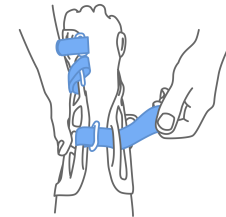
- 2 -

Gently release the hold on both sides of the splint
 NB: Delicately push on both sides of the splint to ensure that it fits properly on the patient's limb.

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- 3 -

Tighten each of the Velcro straps
 The splint must fit the patient perfectly, without pressure points.

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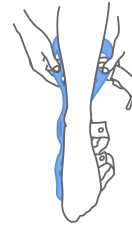
- 1 -

Pull open the splint and place it around the patient's foot

Hold either side of the splint with each hand. Gently pull on both sides. Insert the patient's foot, starting with the heel.

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- 2 -

Gently release your hold on both sides of the splint
 NB: Delicately push on both sides of the splint to ensure that it fits properly on the patient's limb.

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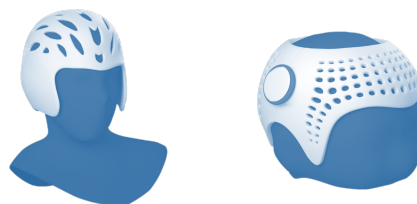
- 1 -

Place the splint correctly on patient's head
 Hold either side of the helmet with each hand. Rest the helmet on the head.

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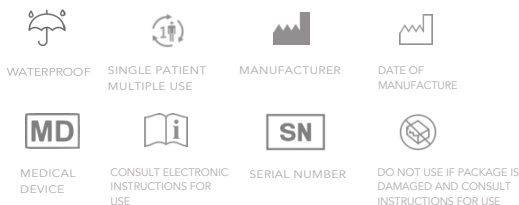
- 2 -

Adjust the headset
 Adjust the headset and secure it with the attachment system under the chin. The headset must fit the patient perfectly, without pressure points.

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- 1 -

Place the mask on the face

Hold either side of the mask with each hand. Gently rest the mask on the face.

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- 2 -

Gently release the hold on the mask

NB: Delicately push on both sides of the mask to ensure that it fits properly on the patient's face.

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- 3 -

Tighten each of the Velcro straps to hold it in place

The mask must fit the patient perfectly, without pressure points.

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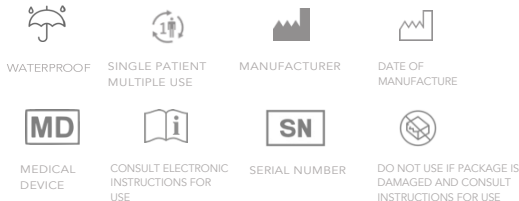
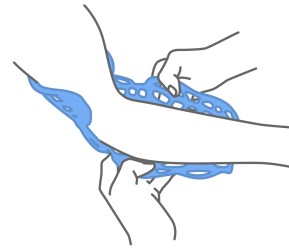
- 1 -

Pull open the splint and place it around the patient's elbow

Hold either side of the splint with each hand. Gently pull on both sides. Insert patient's forearm, the elbow and the arm in the splint while having the arm bent in the shape of the splint.

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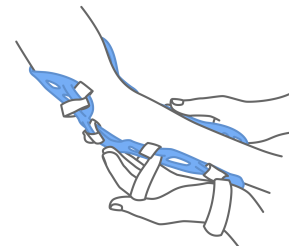
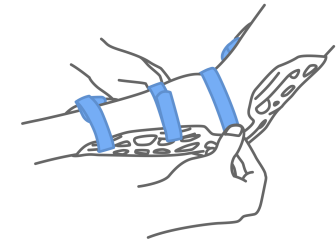
- 2 -

Gently release your hold on both sides of the splint
NB: Delicately push on both sides of the splint to ensure that it fits properly on the patient's limb.

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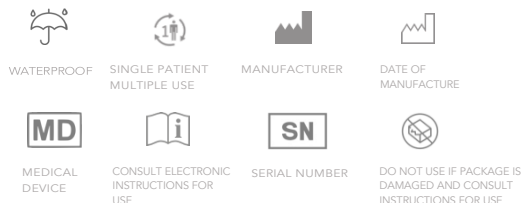
- 3 -

Tighten each of the Velcro straps
The splint must fit the patient perfectly, without pressure points.

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« RX ONLY »

H1: Carpometacarpal Joint INSTRUCTIONS FOR USE

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H2: Thumb – Carpometacarpal Joint INSTRUCTIONS FOR USE

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H3: Hand – Pulpal support for 1, 2 or 3 Fingers INSTRUCTIONS FOR USE

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H4: Hand – Pulpal support for 1, 2 or 3 Fingers INSTRUCTIONS FOR USE

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- 1 -

Expand the splint to place it on the palm/hand of the patient

With each of your hands hold each side of the splint. Gently pull on both sides. Insert the hand of the patient in the splint by starting with the thumb.

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- 2 -

Gently release the hold on both sides of the splint

NB: Delicately push on both sides of the splint to ensure that it fits properly on the patient's limb.

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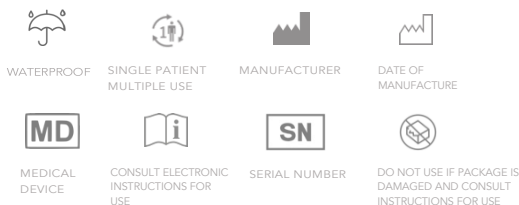
- 1 -

Separate the splint into two halves, and place them around the patient's neck

Hold either side of the splint with each hand. Gently hold them to the patient's neck.

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- 2 -

Gently push both sides of the splint together

NB: Delicately push on both sides of the splint to ensure that it fits properly on the patient's neck.

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- 3 -

Use the bands provided to affix both parts of the splint together

The splint must fit the patient perfectly, without pressure points.

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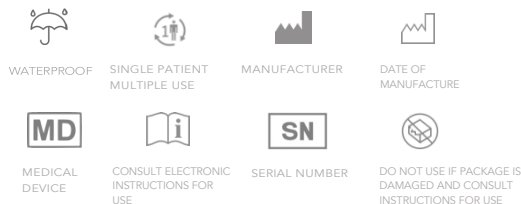
Q2: Thomine – Intrinsic (POSI position) INSTRUCTIONS FOR USE

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Q6: Thomine with individually separated fingers INSTRUCTIONS FOR USE

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- 1 -

Place the patient's forearm correctly on the splint

Place the arm of the patient on the splint by starting with the thumb.

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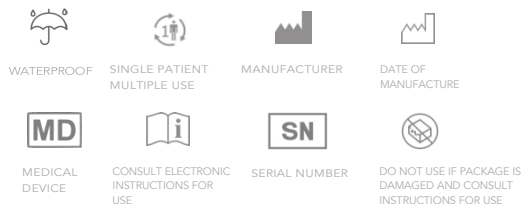
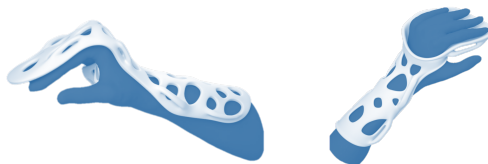
- 1 -

Place the patient's forearm correctly on the splint
Hold either side of the splint with each hand. Gently pull on both sides. Place the hand of the patient on the splint by inserting it between the upper and lower parts of the splint.

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- 2 -

Gently release the hold on both sides of the splint
NB: Delicately push on both sides of the splint to ensure that it fits properly on the patient's limb.

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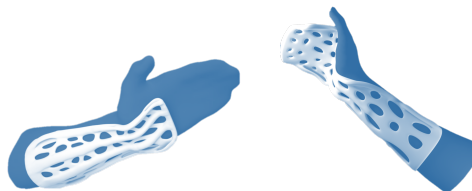
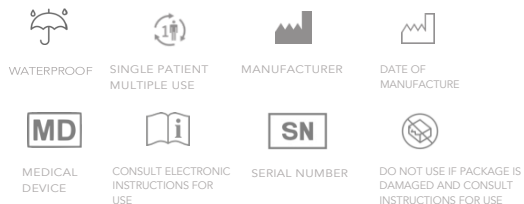
Place the patient's forearm correctly on the splint
 Place the arm of the patient on the splint by starting with the thumb.

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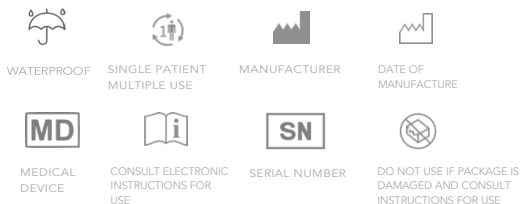
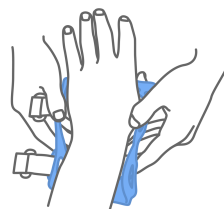
- 1 -

Expand the splint to place it on the hand of the patient
 Hold either side of the splint with each hand. Gently pull on both.

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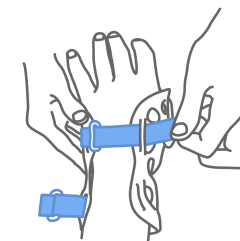
EN. CONTENTS: This device

PRECAUTIONS:

- Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.
- The device is manufactured according to the medical prescription.
- For single patient use only.
- The orthopaedic technician can thermoform the brace. Spentys cannot be held responsible for any non-compliance that may occur as a result of this action.

WARNINGS:

- This device must only be in contact with healthy intact skin. Patients with irritated skin and/or allergies should consult their doctor to know if they can wear this device.
- The medical support splint should not be worn too tightly. This may apply excessive mechanical force or shear forces on the skin.
- Keep this device away from open flames and other heat sources as this may lead to serious burns and injuries.
- If you experience, or have an increase in pain, redness, swelling, skin irritation, sensitization, allergy, immune response, or any adverse reactions in the skin where the device has contact while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- This device is not intended to prevent injury, or to reduce or eliminate the risk of further injury.

CONTRAINDICATIONS AND POTENTIAL COMPLICATIONS:

- Do not use this device if it was not prescribed by your physician.
- Do not use this device if you have a known allergy to plastic, if you have high sensitive skin, or if you have any kind of open-scars.
- This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CARE AND WASHING INSTRUCTIONS:

- Hand wash in cold water with a mild soap, rinse and lay flat to air dry.

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