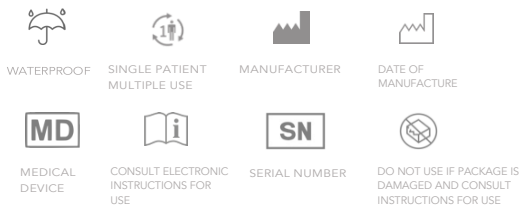
INSTRUCTIONS FOR USE

« RX ONLY »

CUSTOM-MADE MEDICAL DEVICE

**EN INTENDED USE:** The Spentys PolyCast is a custom-made medical device, intended for immobilization of a patient's limb. As an orthopedic medical support orthosis, it is used to heal bones, muscles or ligaments by immobilizing and supporting the limb during a specific period of time. The orthopedist needs to ensure that the limb keeps the same position during the whole healing period. The healing period is indicated by the physicians. The intended use of each individual PolyCast is defined by the prescribing physician.

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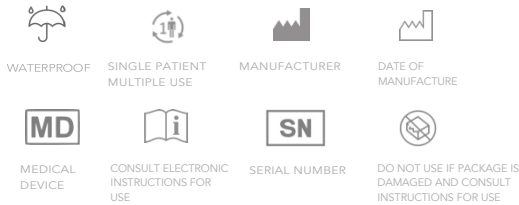
« RX ONLY »

CUSTOM-MADE MEDICAL DEVICE

**A1: Forearm – Dorsal Opening INSTRUCTIONS FOR USE**

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**A3: Forearm – Medial Opening INSTRUCTIONS FOR USE**

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**A4: Forearm & Thumb – Medial Opening INSTRUCTIONS FOR USE**

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- 1 -

Expand the orthosis to place it on the arm of the patient  
Hold either side of the orthosis with each hand. Gently pull on both sides. Insert the arm of the patient in the orthosis, starting with the thumb.

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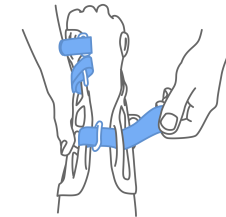
- 2 -

Gently release the hold on both sides of the orthosis  
NB: Delicately push on both sides of the orthosis to ensure that it fits properly on the patient's limb.

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
- 3 -

Tighten each of the Velcro straps  
The orthosis must fit the patient perfectly, without pressure points.

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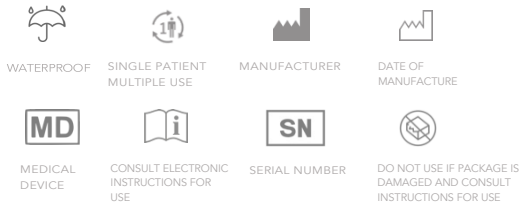« RX ONLY »

## CUSTOM-MADE MEDICAL DEVICE

B1: Foot – Immobilization Boot INSTRUCTIONS FOR USE

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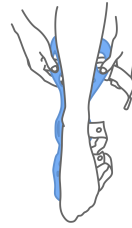
B8: Foot - Articulated AFO INSTRUCTIONS FOR USE

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- 1 -

Pull open the orthosis and place it around the patient's  
foot

Hold either side of the orthosis with each hand. Gently pull on both  
sides. Insert the patient's foot, starting with the heel.

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- 2 -

Gently release your hold on both sides of the orthosis  
NB: Delicately push on both sides of the orthosis to ensure that it  
fits properly on the patient's limb.

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- 3 -

Tighten each of the Velcro straps

The orthosis must fit the patient perfectly, without pressure points.

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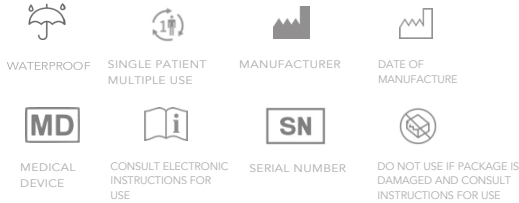
- 1 -

Place the mask on the face

Hold either side of the mask with each hand. Gently rest the mask on the face.

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- 2 -

Gently release the hold on the mask

NB: Delicately push on both sides of the mask to ensure that it fits properly on the patient's face.

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- 3 -

Tighten each of the Velcro straps to hold it in place

The mask must fit the patient perfectly, without pressure points.

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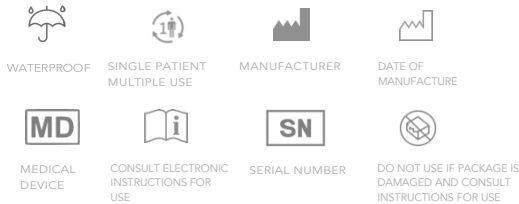
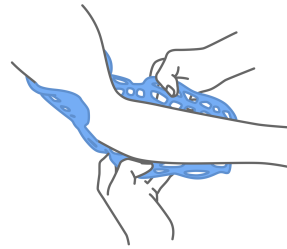
E6: Elbow - articulated INSTRUCTIONS FOR USE

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- 1 -

Pull open the orthosis and place it around the patient's elbow

Hold either side of the orthosis with each hand. Gently pull on both sides. Insert patient's forearm, the elbow and the arm in the orthosis while having the arm bent in the shape of the orthosis.

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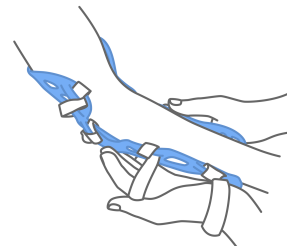
5

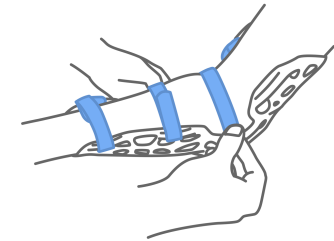
- 2 -

Gently release your hold on both sides of the orthosis  
NB: Delicately push on both sides of the orthosis to ensure that it fits properly on the patient's limb.

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« RX ONLY »

CUSTOM-MADE MEDICAL DEVICE

H1: Carpometacarpal Joint INSTRUCTIONS FOR USE

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H2: Thumb – Carpometacarpal Joint INSTRUCTIONS FOR USE

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H3: Hand – Pulpal support for 1, 2 or 3 Fingers INSTRUCTIONS FOR USE

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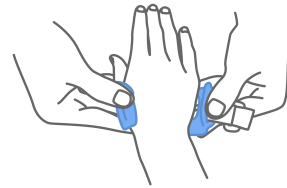
H4: Hand – Pulpal support for 1, 2 or 3 Fingers INSTRUCTIONS FOR USE

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- 1 -

Expand the orthosis to place it on the palm/hand of the patient

With each of your hands hold each side of the orthosis. Gently pull on both sides. Insert the hand of the patient in the orthosis by starting with the thumb.

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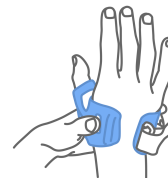
- 2 -

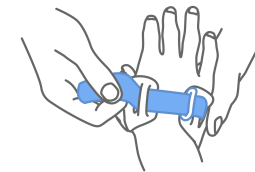
Gently release the hold on both sides of the orthosis

NB: Delicately push on both sides of the orthosis to ensure that it fits properly on the patient's limb.

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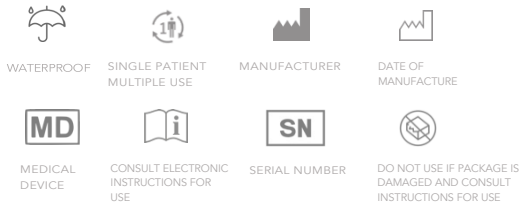
- 1 -

Expand the orthosis to place it on the hand of the patient

Place the patient's on the orthosis by starting with the thumb.

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Expand the orthosis to place it on the arm of the patient  
 Hold either side of the orthosis with each hand. Gently pull on both sides. Insert the arm of the patient in the orthosis, starting with the thumb. Then place correctly the two fingers that have to be immobilized.

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- 2 -

Gently release the hold on both sides of the orthosis  
 NB: Delicately push on both sides of the orthosis to ensure that it fits properly on the patient's limb.

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- 1 -

Place the patient's forearm correctly on the orthosis  
 Hold either side of the orthosis with each hand. Gently pull on both sides. Place the hand of the patient on the orthosis by inserting it between the upper and lower parts of the orthosis.

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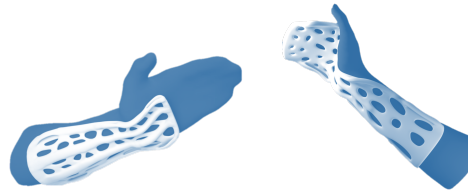
- 3 -

Tighten each of the Velcro straps  
 The orthosis must fit the patient perfectly, without pressure points.

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Place the patient's forearm correctly on the orthosis  
Place the arm of the patient on the orthosis by starting with the thumb.

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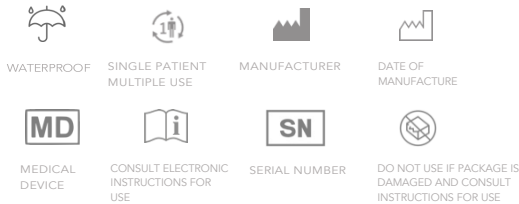
« RX ONLY »

CUSTOM-MADE MEDICAL DEVICE

W1: Wrist - Dorsal Opening INSTRUCTIONS FOR USE

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W2: Wrist & Thumb - Dorsal Opening INSTRUCTIONS FOR USE

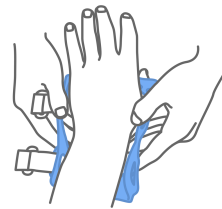
- W2: Poignet et pouce - Ouverture dorsale NOTICE D'UTILISATION
- W2: Pols en duim - Dorsale opening INSTRUCTIES VOOR GEBRUIK
- W2: Handgelenk & Daumen - Dorsale Öffnung GEBRAUCHSANWEISUNG

W3: Wrist - Medial Opening INSTRUCTIONS FOR USE

- W3: Poignet - Ouverture médiale NOTICE D'UTILISATION
- W3: Pols - mediale opening INSTRUCTIES VOOR GEBRUIK
- W3: Handgelenk - Mediale Öffnung GEBRAUCHSANWEISUNG

W4: Wrist & Thumb - Medial Opening INSTRUCTIONS FOR USE

- W4: Poignet et pouce - Ouverture médiale NOTICE D'UTILISATION
- W4: Pols en duim - mediale opening INSTRUCTIES VOOR GEBRUIK
- W4: Handgelenk & Daumen - Mediale Öffnung GEBRAUCHSANWEISUNG

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Hold either side of the orthosis with each hand. Gently pull on both.

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## EN. CONTENTS: This device

### PRECAUTIONS:

- Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.
- The device is manufactured according to the professional medical data shared by the physician based on the medical prescription. The physician must provide the required data about the patient to correctly model the patient's PolyCast; all data is anonymized.
- Users of the Spentys PolyCast solution are physicians that need to assure the immobilization of their patient's limb. A physician should always choose a specific device for a specific patient based on their clinical judgement.
- For single patient use only.
- The orthopaedic technician can thermoform the brace. Spentys cannot be held responsible for any non-compliance that may occur as a result of this action.
- Avoid exposure to direct sunlight or hot environments for a long time, this might affect the mechanical properties of the device.
- Orthoses may lead to impaired driving performance.
- Blisters may occur in areas of high local stresses (e.g. ulnar head).
- Oedematous, inflamed regions might protrude through the lattice structure, potentially causing discomfort for the patient.
- Wearing the device for a longer period than recommended by the physician can potentially be detrimental for the patient, resulting in potential splint dependency, stiffness, atrophy and joint disuse.

### WARNINGS:

- This device must only be in contact with healthy intact skin. Patients with irritated skin and/or allergies should consult their doctor to know if they can wear this device.
- The medical support splint should not be worn too tightly. This may apply excessive mechanical force or shear forces on the skin.
- Keep this device away from open flames and other heat sources as this may lead to serious burns and injuries.
- If you experience, or have an increase in pain, redness, swelling, skin irritation, sensitization, allergy, immune response, or any adverse reactions in the skin where the device has contact while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- This device is not intended to prevent injury, or to reduce or eliminate the risk of further injury.
- After the healing period defined by the physician, please throw the PolyCast away in the plastic bin to avoid other people using it inappropriately.

### CONTRAINDICATIONS AND POTENTIAL COMPLICATIONS:

- Do not use this device if it was not prescribed by your physician.
- Do not use this device if you have a known allergy to polymer, if you have high sensitive skin, or if you have any kind of open-scars.
- This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

### CARE AND WASHING INSTRUCTIONS:

- Hand wash in cold water with a mild soap, rinse and lay flat to air dry.

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